

To Screen Patients

TO SCREEN PATIENTS	
A. To Add New Patient	
1. Select the View All Patients tab.	Home Users View all patients Role management Plan purchase Reports Sites How It Works Educational materials
 Click the Add New button. The screen will refresh to display the Add New Patient window. 	First Name Last ## Phone ## SSN DOB Gender Most Recent Test Date Most Recent Test Score Action Ernest Abcde 4321 05/21/1954 M 04/28/2013 32 Select J Edit
3. Enter the information requested, including the primary physician for the patient, and click on the Register button.	Register
B. To Order Screen	
1. Select the tab labeled View All Patients and select a patient from the list.	Home Users View all patients Role management Plan purchase Reports Sites How It Works Educational materials
2. To begin the screening assessment, click on the Start New Screen button located in the top right of the M3 Patient Report page.	Action START NEW SCREEN
3. You will be prompted to choose if you would like to administer any additional supplemental questionnaires. You may also be prompted to select the site where the screen is being administered.	This action will use a purchased test code for the screening process. Do you want to continue? If M3 Screen(Validated for 18 years old and up)
	Check the additional questionnaire(s) to be added for the screening Side effect questionnaire Audit C questionnaire PHQ-9 questionnaire Supplemental questions Select the Site where screening is to be assigned Select
4. Click Continue to administer the screen.	CONTINUE
C. For Patient to Take the Screen	
 From the M3 Clinician Portal home page, or the tablet screening device, select the Start Screen action button in your desired language. 	Start Screen/Pantalla Inicio English Spanish
 Within the Authenticate window the patient will enter the last four digits of their Member ID number, their last name and their date of birth, and then click on the Start Screen 	Last 4 digits of SSN*
button.	Last Name*
	Date Of Birth*
	Enter date as mm/dd/yyyy
3. Instruct the patient to read and click Accept on the medical disclaimer window to continue with the screen.	
4. The patient will now be administered the screen and may proceed with the checklist.	ACCEPT